



Micro Direct, Inc.  
803 Webster Street  
Lewiston, ME 04240  
(800) 588-3381  
(207) 786-7280 FAX  
www.mdspiro.com

## Breath Hydrogen Monitor Cleaning Recommendations

Using a SafeBreath (order #FM200) not only protects your patient from potential cross infection during testing but also protects and increases the life of your fuel cell and eliminates the need for and cleaning of the plastic mouthpiece adapter.

### External Surfaces

The external housing of the H2 Check, other than the sensor surface, may be wiped with any sterile non-alcohol wipe or a damp cloth. It is recommended that this procedure is performed after each use and all used cardboard mouthpieces discarded.

**IMPORTANT NOTE:** The sensor surface must NOT be wiped with any aqueous solutions and must not be exposed to solvents e.g. alcohol or permanent damage may result.

If you use a regular cardboard mouthpiece with the mouthpiece adapter, follow the instructions below.

### Mouthpiece Adapter Cleaning Recommendations

The plastic adapters should be replaced on a monthly basis or after 250 tests (whichever is sooner). Between tests the adapter may be immersed in a cold disinfection solution or mild detergent sterilization solution then **MUST** be thoroughly rinsed in water and dried completely before reuse.