

# Spirotrac 6 Local Installation



Vitalograph - Spirotrac Installation



## Spirotrac

- [Install Spirotrac](#)
- [Launch Spirotrac IFU](#)
- [View Spirotrac Release Notes](#)

## Database Studio

- [Install Database Studio](#)

## Other Applications

- [Install SQL Server](#)
- [Install Connect](#)
- [View Connect Release Notes](#)
- [Install Teamviewer](#)

## Drivers

- [Install USB Pneumotrac](#)

## Other Manuals

- [Launch Connect IFU](#)

## Spirotrac Installation

Welcome to the Vitalograph Spirotrac Installer.

Administrator rights are required to install Spirotrac.


Components which will be installed include Microsoft SQL Server Express and the Microsoft .NET Framework Versions 4.5, if not already installed.

For further information or assistance contact [technical.support@vitalograph.ie](mailto:technical.support@vitalograph.ie)


Copyright © 2021 Vitalograph

Select Install Spirotrac


# Spirotrac 6 Local Installation

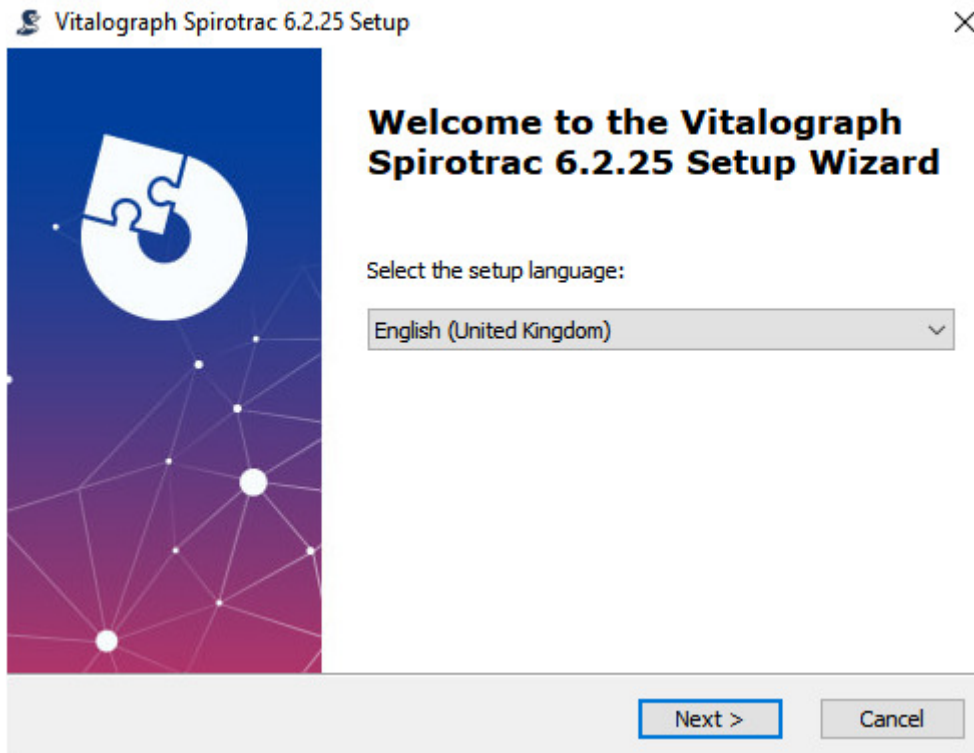


Select language – Next

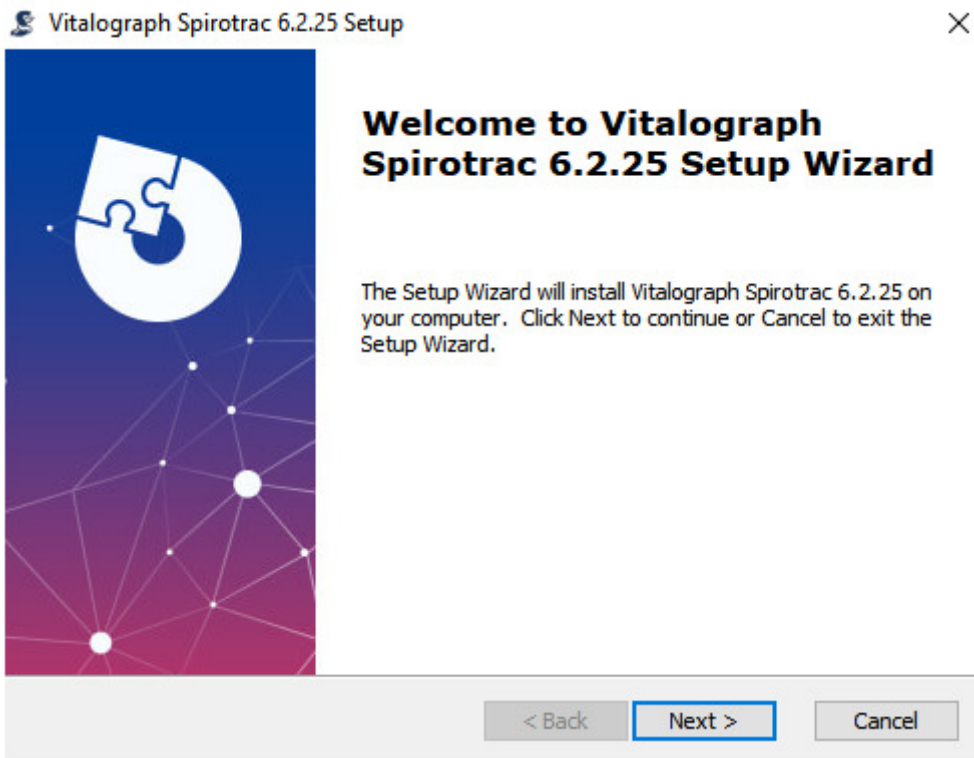

Select Next

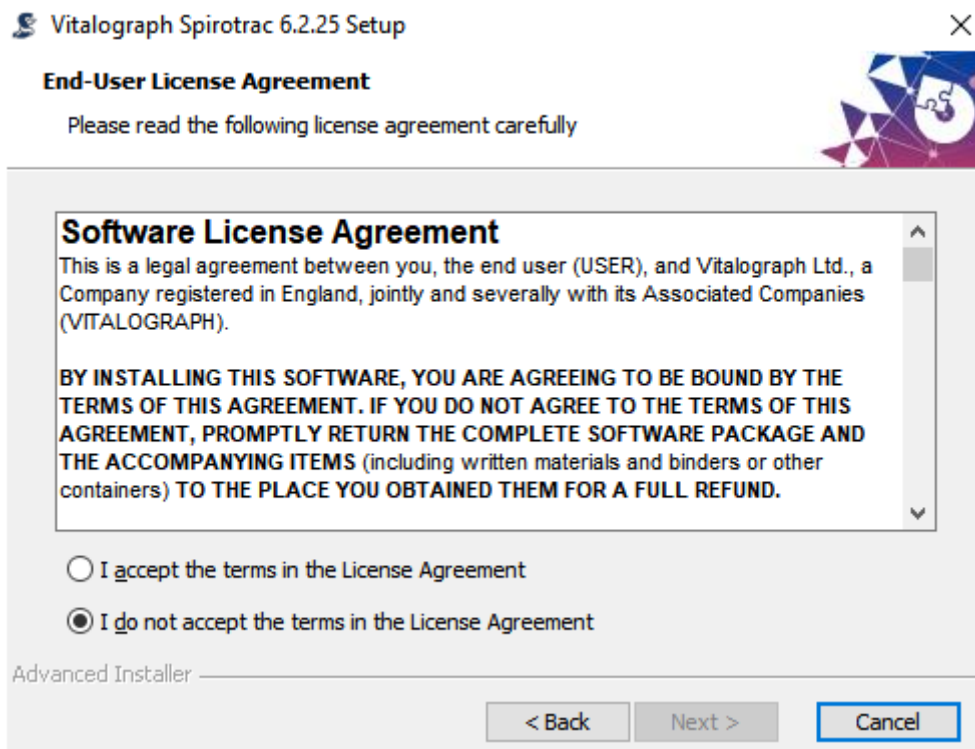


Review the Software License Agreement – Select “I accept the terms in the Licence Agreement”

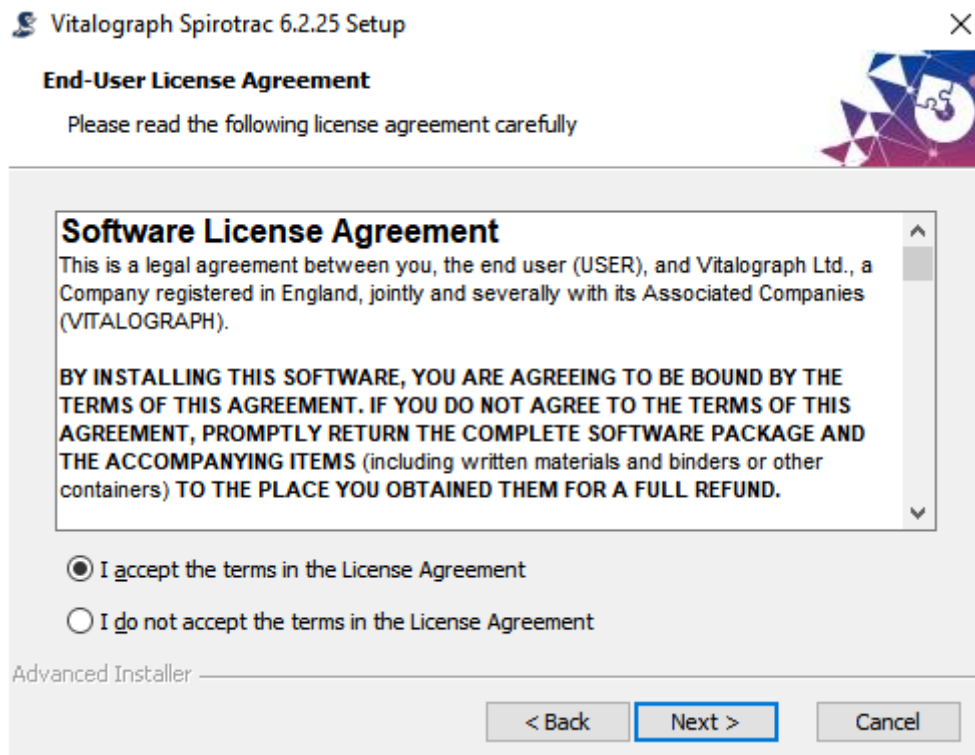


# Spirotrac 6 Network Installation



## Select Next

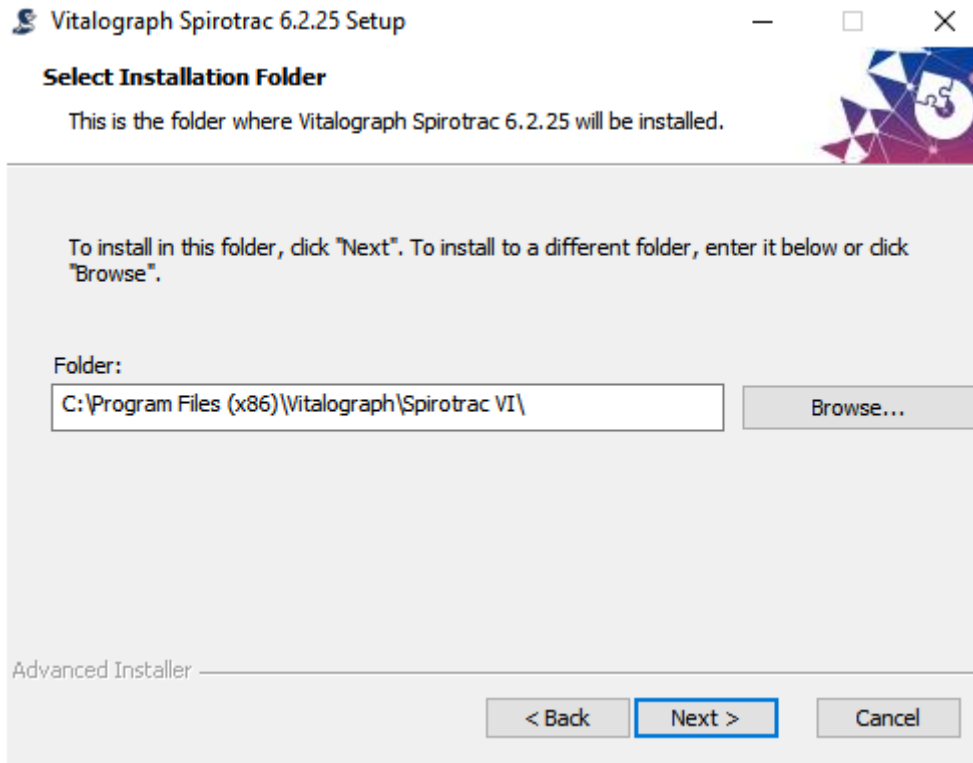


## Select Next

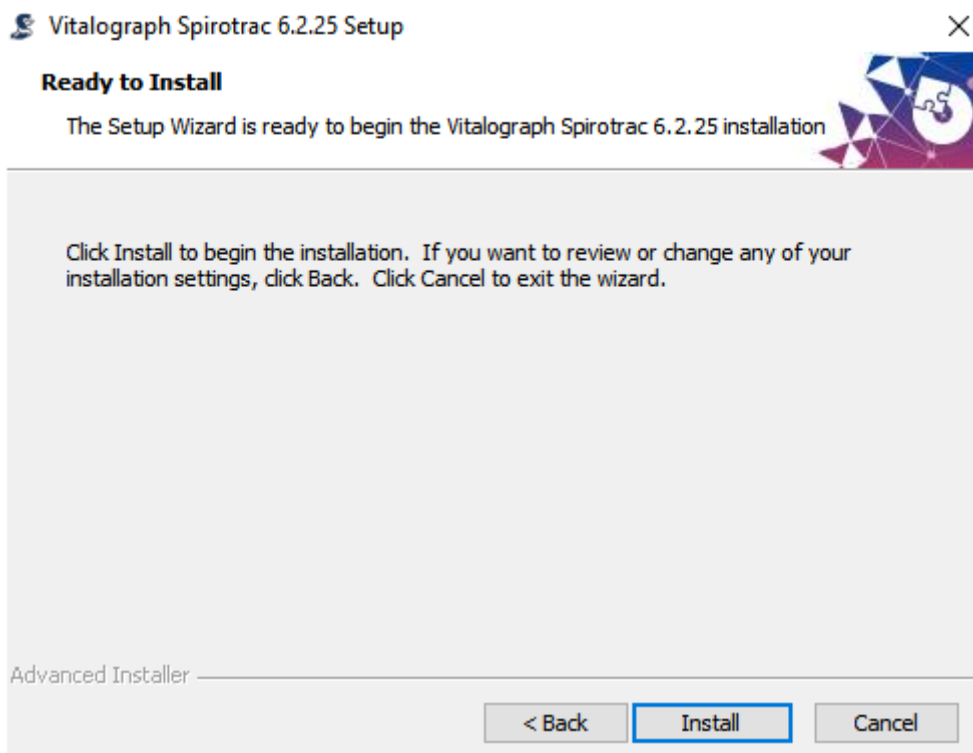


# Spirotrac 6 Local Installation



## Select Install

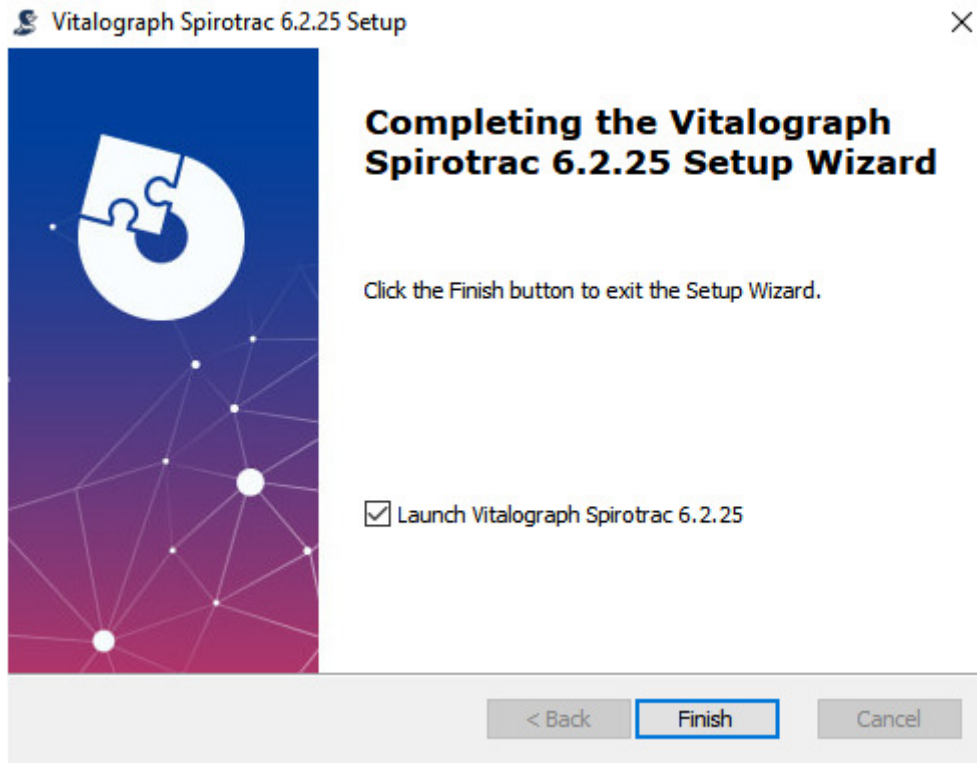

## Select Finish

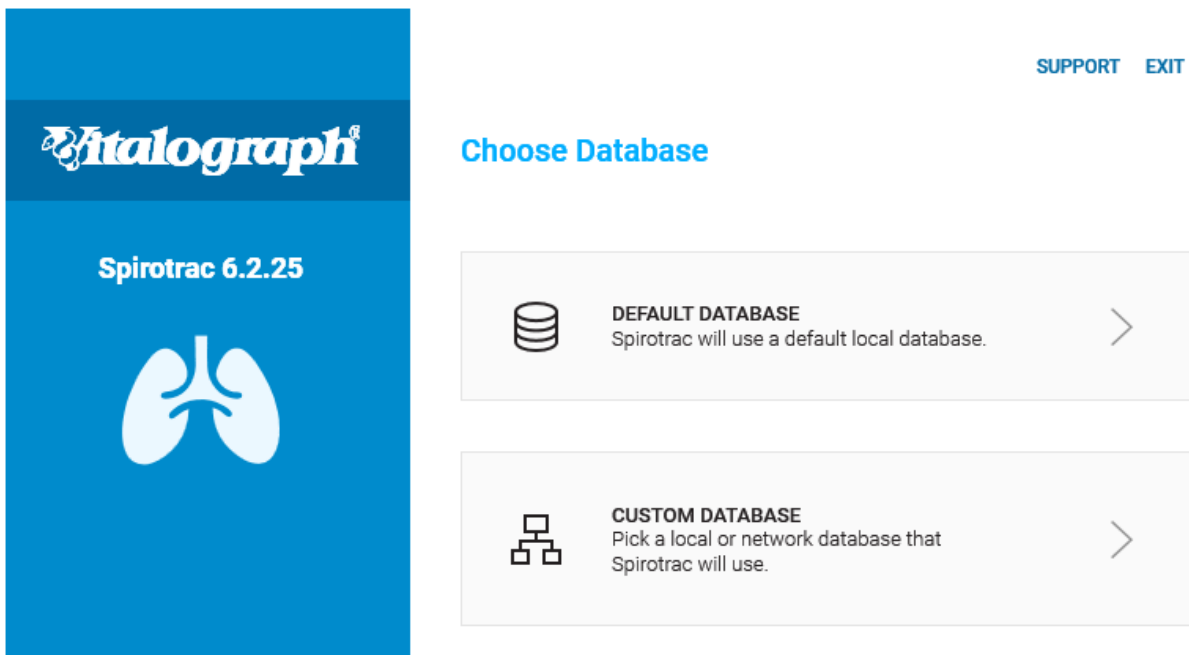

## Select DEFAULT DATABASE

# Spirotrac 6 Local Installation



**atalograph** Spirotrac - Setup

**Site**

Name

The name will appear on all reports.

**Administrator**

Username

Security Questions

Answer

Password

Confirm Password

Password must contain at least 8 characters, including UPPER, lowercase and number.

**Welcome**

Please take a moment to set up your instance of Spirotrac.

Enter details

**atalograph** Spirotrac - Setup

**Site**

Name

The name will appear on all reports.

**Administrator**

Username

Security Questions

Answer

Password


Confirm Password

Password must contain at least 8 characters, including UPPER, lowercase and number.

**Welcome**

Please take a moment to set up your instance of Spirotrac.

Select Continue

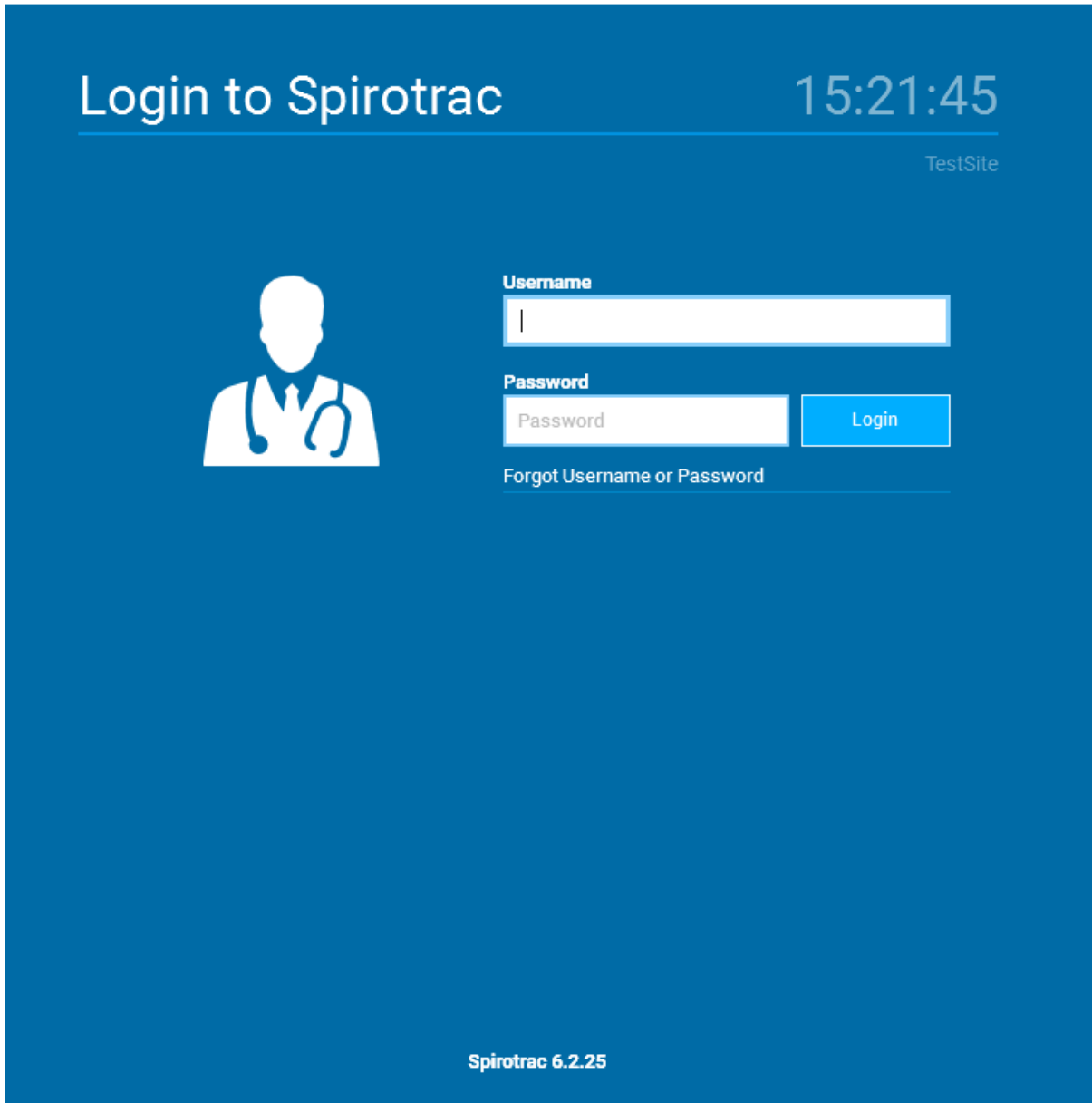

The screenshot shows a login page with a blue background. At the top left, the text "Login to Spirotrac" is displayed in white, underlined. To the right, the time "15:21:45" is shown in white. Below the time, the text "TestSite" is visible in a smaller font. On the left side, there is a white icon of a person wearing a stethoscope. To the right of the icon, there are two input fields: "Username" and "Password". The "Username" field contains a single vertical bar. Below the "Password" field is a blue "Login" button. Underneath the "Password" field, there is a link that says "Forgot Username or Password". At the bottom center of the page, the text "Spirotrac 6.2.25" is displayed in white.

Login to Spirotrac 6

# Spirotrac 6 Local Installation



A screenshot of the Spirotrac 6.2.25 software interface. The window title bar shows 'Spirotrac 6.2.25' and the system clock '15:22:22 11/03/2021'. The interface has a dark blue header with a search bar containing 'Subject' and a plus sign. A left sidebar contains a menu with items: HOME, SETTINGS, REPORTS, AUDIT LOG, HELP, ABOUT, LOCK, LOG OFF, and EXIT. The main content area is a light blue gradient with the text 'Welcome to Spirotrac' and 'GET STARTED QUICKLY'. Below this are two buttons: 'SELECT SUBJECT' with a magnifying glass icon and 'ADD SUBJECT' with a plus sign icon. A 'CALIBRATION VERIFICATION' section is partially visible in the sidebar. At the bottom, a red banner reads 'NOT ACTIVATED' with a message: 'To activate your copy of Spirotrac and unlock all features, click on the license icon in the main menu at any time.'